

INTELEVIEWER SOFTWARE DOWNLOAD

1. Select from the following options under 'Referrers' page of the Radius Imaging website or type in the address:
<https://radiusimaging.com.au/referrers>



2. **Run** the application
3. Select **Edit Bookmarks** to enter Radius server details



4. Click **Add**
5. In **Server URL** enter:
<https://images.radiusimaging.com.au>



6. Click **Use Custom Server Name** and type in 'Radius Imaging'



7. Click **OK** and Close

8. **Login** using your Username and Password